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Instructions for use

Please read this information carefully because it will help you to use HYLO PARIN® properly.

HYLO PARIN®

Sodium hyaluronate and heparin sodium

For moisturisation and care of cornea and conjunctiva in irritated eyes with hyperemia, burning and itching

Sterile, phosphate-free eye drops without preservatives

What is HYLO PARIN®?

HYLO PARIN® is a sterile, phosphate- and preservative-free solution for ocular use.

Composition:

1 ml HYLO PARIN® contains 1 mg sodium hyaluronate, 1300 I.U. heparin sodium (from the intestine mucosa of pigs), glycerol, citric acid anhydrous, sodium citrate and water for injections.

What is HYLO PARIN® used for?

HYLO PARIN® humidifies, moistens and cares the cornea and conjunctiva and relieves the irritations of the eye caused by environmental burden. Symptoms such as redness of the eye, burning and itching are relieved.

When should HYLO PARIN® be used?


Burning, itching, tearing, foreign body sensation or the sensation of dry eyes are symptoms of an irritated eye. You rub your eyes but the complaints persist. This is a symptom that the eye is not sufficiently provided with humidity. The surface of a healthy eye is covered by a smooth, even film of tears which keeps the tissues fully functioning and rinses off any particles of dust or dirt. Some conditions put extra burden on the eyes and reduce the physiologic function of the tears.

This may be caused by staying in air-conditioned rooms, prolonged computer work or staying in a dusty environment. With increasing age less tears are produced.

HYLO PARIN® contains sodium hyaluronate, a physiologic substance which is found in the eye but also in other tissues of the body. Sodium hyaluronate has unique properties which allow it to form an even, stable, especially long-lasting layer of humidity on the surface of the eye without being rinsed away quickly. This so-called bio-adhesive property is supported by heparin. Heparin also is a physiologic substance which resembles in its chemical structure sodium hyaluronate. Sodium hyaluronate and heparin care for the cornea and conjunctiva and together with the physiologic tears they lubricate the surface of the eye. The eye is supplied with a long-lasting and stable tear film and by this protected against environmental irritations. Being preservative-free and containing physiologic substances HYLO PARIN® is usually very well tolerated.

How should HYLO PARIN® be applied?


Please do not use HYLO PARIN® if the seal on the folding carton is broken or damaged before using for the first time (☹).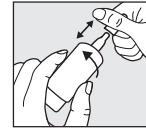

Step 1:

Remove cap before use.

To do this, hold the cap above the bottle. With your other hand, pull away the bottle from the cap with a rotary movement. Before the first application please turn the HYLO PARIN® bottle with its tip down and press the bottom of the bottle repeatedly until the first drop appears at the nozzle. Now the bottle is ready for further use.

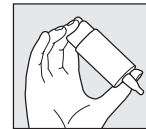
Step 2:

Hold the bottle upside down and place your thumb on the shoulder of the bottle and the other fingers on the base of the bottle.


Step 3:

Support the hand holding the COMOD®-bottle with your other hand as shown.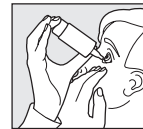

Step 4:

With your head leaning back slightly, gently pull down your lower eyelid. Press the base of the bottle vigorously and quickly. This activates the mechanism for the release of one drop. Because of the special COMOD® valve system, the size and speed of each drop remain constant regardless of the pressure applied to the base of the bottle. Slowly close your eyes to allow the fluid to spread evenly on the eye surface.

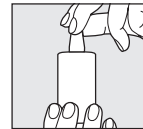Step 5:

Replace cap immediately after use. Ensure that the dropper tip is dry. During use, please avoid any contact between the tip of the bottle and your eye or your skin.

Each bottle releases 10 ml of solution which is equivalent to approximately 300 drops. For technical reasons a small amount will remain in the bottle.

HYLO PARIN® can be used for an unlimited period of time.

HYLO PARIN® is only for you, even people with similar symptoms should not use your eye drops.

What else do you need to know?

If you use any other eye drops there should be at least 30 minutes between the applications, and HYLO PARIN® should always be used last. Eye ointments should, however, always be administered after using HYLO PARIN®.

Because it is preservative-free HYLO PARIN® is well tolerated even when used over a longer period of time.

In very rare cases hypersensitive reactions like burning or excessive tearing have been reported which stopped immediately when the use of HYLO PARIN® was discontinued. In very rare cases allergic reactions of the skin to heparin have been reported.

HYLO PARIN® may be used while wearing contact lenses. After inserting contact lenses, waiting for about 30 minutes before applying the eye drops is recommended. In doing so you avoid intolerance reactions which may occur due to interactions between lens care products and HYLO PARIN®.

Based on the available evidence HYLO PARIN® may be used in children and pregnant or lactating women.

If gentamicin-containing eye drops should be used concomitantly please leave a long interval of at least one hour between both applications as gentamicin in direct contact with heparin may form visible precipitates in the conjunctival sac.

How often and for how long can you use HYLO PARIN®?

HYLO PARIN® should be used according to the instructions given by your doctor. Generally, you instil one drop of HYLO PARIN® three to six times a day into the conjunctival sac of each eye. If you apply HYLO PARIN® more frequently (e.g. more than 10 times per day) or suffer from more severe complaints you should consult an ophthalmologist regularly. HYLO PARIN® is suitable for long-term treatment. However – as in all persisting symptoms – you should consult your ophthalmologist if your complaints do not improve e. g. within two weeks.

When should you not use HYLO PARIN®?

HYLO PARIN® should not be used if you are hypersensitive to one of the ingredients.

What you need to know about storing HYLO PARIN®?

Do not store above 25 °C (↘^{25 °C}).

HYLO PARIN® can be used for 6 months after opening.

Do not use beyond expiry date (☞).

Keep out of the reach of children.

The following package sizes are available:

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STERILE A

Medical device

Your opinion about our product is relevant for us!
We are looking forward to get your feedback via mail or
via www.ursapharm.info/hyloparin.

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URSAPHARM Arzneimittel GmbH,